

M 7000

**INSTRUCTION FOR
MULTI-FUNCTION SYSTEM**



INSTRUCTIONS FOR AC-2070 MULTI-FUNCTION SYSTEM

ATTENTION: ONLY STATE-LICENSED AND PROFESSIONAL ESTHETICIANS ARE ALLOWED TO OPERATE THIS SKINCARE SYSTEM. ANY VIOLATIONS OF THE ABOVE WILL VOID THE WARRANTY AND THE OPERATOR SHALL BE RESPONSIBLE FOR ANY DAMAGES TO THE SYSTEMS AND/OR CLIENTS.

INSTALLATION :

1/TAKE OUT THE BASE OF THE SYSTEM WITH THE CONTROL PANEL FACING YOU.

2/PUT THE STEAMER OUT FROM THE BOX AND INSERT THE TO THE BIGGER HOLDER ON THE LEFT SIDE OF THE SYSTEM. CONNECT THE POWER CORD OF THE STEAMER TO THE LSFT SIDE OF THE CENTRE COLUMN.

3/CONNECT THE MAIN POWER CORD FROM THE BASE TO THE POWER OUTLET. THIS MAIN POWER CORD COMES WITH THE ACCESSORIES.

4/UNWRAP THE MAGNIFYING LAMP AND INSERT IT TO THE RIGHT SIDE SHAFT OF THE SYSTEM. CONNECT THE POWER CORD OF THE MAGNIFYING LAMP TO THE RIGHT SIDE OF THE CENTRE COLUMN.

5/SWITCH ON THE MAIN POWER AND CHECK WHETHER ALL THE FUNCTION LIGHTING ARE ON. THE STRENGTH AND INTENSITY OF EACH FUNCTION IS CONTROLLED BY THE DIAL ON THE CONTROL PANEL.

6/TAKE OUT THE ACCESSORY TRAY AND INSTALL IN FRONT OF THE SYSTEM BY INSERTING THE STRAIGHT TUBINGS INTO THE TWO HOLES IN FRONT OF THE SYSTEM.

7/LOOK FOR THE BLACK HI-FREQUENCY HANDLE, LIKE A GIANT PENCIL SHAPE AND CONNECT TO THE APPROPRIATE POSITION AT THE SECOND RIGHT UNDER THE CONTROL PANEL, SEE LABEL.

8/TAKE OUT THE WHITE BRUSH HANDLE AND CONNECT TO THE SECOND PLUG FROM THE LEFT, SEE LABEL. INSERT THE DESIRED BRUSH INTO THE SMALL OPENING OF THE HANDLE AND PUSHING IN SLIGHTLY TO SECURE THE POSITION OF THE BRUSH.

9/EXAMINE THE TWO HOSES OF THE VACUUM AND SPRAY. SWITCH ON AND FEEL THE SUCTION AND THE SPRAY. FOR THE SUCTION HOSE, YOU CAN CONNECT ANY OF THE THREE SMALL PIECES OF GLASS VENTHOSES PROVIDED SO THAT YOU CAN OPERATE THE VACUUM FUNCTION. FOR THE HOSE YOU FEEL AIR COMING OUT, CONNECT TO EITHER ONE OF THE BOTTLES PROVIDED. YOU SHOULD FILL IN THE BOTTLES WITH DESIRED LOTIONS. YOU WILL THEN FEEL THAT THERE IS A PIN HOLE AT THE SIDE OF THE CHROME CAP OF THE BOTTLE WITH A STREAM OF AIR COMING OUT. THIS IS NORMAL. LOTION WILL SPRAY WHEN YOU PUT YOUR FINGER TIP OVER THE TOP OF CHROME BOTTLE AND HOLDING TIGHT FOR THE MOMENT. IT IS VERY IMPORTANT TO KEEP THESE BOTTLE HEAD ALWAYS CLEAN AND WITHOUT ANY BLOCKAGE FOR THE SPRAYING PIN HOLE. IT IS SUGGESTED TO CLEAN IT EVERY DAY WITH WARM WATER AND SOAPING WITH ALCOHOL TO DISSOLVE ANY RESIDUES IN THE BOTTLE HEAD. FOR THE PIN HOLE WHERE LOTIONS ARE SPRAYING OUT, CLEAN IT WITH A PIN OR NEEDLE IF NEEDED.

10/TAKE OUT THE GALVANIC ACCESSORIES, LOOK FOR THE SPECIAL CORD WITH TWO TERMINUS--BLACK AND RED WITH PIN PLUG AT THE ENDS. FOR THE 7-PIN PLUG, CONNECT TO THE FIRST PULG UNDER THE CONTROL PANEL. AFTER CONNECTED SECURELY, LOOK FOR THE 4-INCHES CHROME ROD COME WITH ONE HOLE AT ONE OF THE ENDS. CONNECT THIS END TO THE PIN OF THE BLACK TERMINAL. LOOK FOR THE OTHE IVORY COLOR HANDLE WITH ONE END WITH A HOLE AND THE TOP WITH A CHROME CONE LIKE SCREW. CONNECT THE RED PIN TO THE BOTTOM OF THE IVORY COLOR HANDLE SECURELY. TWIST THE CONE LIKE SCREW ANTI-CLOCKWSE AND YOU CAN INSERT ANY OF THE PROVIDED CHROME ACCESORIES FOR GALVANIC LIKE THE ROLLER, FLAT HEAD OR BULLET HEAD AND DOUBLE BALLS FOR YOUR OPERATION OF GALVANIC OVER DIFFERENT PART OF THE FACE. YOU MAY CONSULT YOUR SCHOOL FOR WHERE TO USE WHICH ACCESORIES OVER THE DIFFERENT PART OF THE FACE.

11/TAKE OUT THE WOODS LAMP. PLUG TO THE RIGHT SIDE OF THE SYSTEM WHERE THE H.F AND GALVANIC ARE ATTACHING TO.

FACIAL STEAMER OPERATION MAMUAL:

1/ MAKE SURE THERE IS NOT ANY PARTICLES OR DUST IN THE JAR.

2/FILL UP THE JAR WITH DISTILLED WATER ONLY THROUGH THE FUNNEL AT THE TOP OF THE STEAMER. MAKE SURE ONLY DISTILLED WATER IS USED, OTHERWISE THE WARRANTY FOR THE STEAMER WILL BE VOIDED. FILL UP TO THE UPPER RED LINE OF THE JAR. DO NOT OVER FILLED THE JAR BECAUSE IT MAY CAUSE SPITTING OF HOT WATER THROUGH THE NOZZLE WHERE STEAM IS COMING OUT AND WILL BURN THE CLIENT SERIOUSLY. AFTER FILLING THE WATER, CLOSE THE PLASTIC LID.

3/TURN ON THE TIMER CLOCKWISELY TO DESIRED TIME. IF THE WATER IS AT ROOM TEMPERATURE, ALLOW 7 MORE MINUTES TO WARM UP THE WATER TO BOILING AND GENERATING THE FIRST STEAM. SO ADD 7 MINUTES TO YOUR TREATMENT TIME.

4/AFTER SEEING THE FIRST STEAM COMING OUT, YOU MAY NOW SWITCH ON THE OZONE LAMP IF DESIRED. YOU MAY FIND THE SWITCH NEXT TO THE TIMER CONTROL. WITH THE OZONE LAMP ON, YOU WILL SEE MORE FINE AND MISTY STEAM BEAM.

5/THE STEAMER WILL OPERATE UNTIL THE TIMER WILL STOP IT OR THE WATER LEVEL IS AS LOW AS THE LOWER RED LINE.

6/CHECK WATER LEVEL IF YOU SUSPECT WATER IS TOO LOW. DO NOT RE-START THE STEAMER WITHOUT ADDING NEW COOL DISTILLED WATER, DOING THIS MAY DAMAGE THE HEATING ELEMENT AND TEMPERATURE CONTROL IMMEDIATELY. IF STEAM IS NEEDED IMMEDIATELY, ADD HOT DISTILLED WATER TO THE WATER JAR.

7/SUGGESTED OPERATION SET UP:

<u>SKIN CONDITION</u>	<u>DISTANCE/IN</u>	<u>TIME/MIN</u>
OILY SKIN	8	15
NORMAL SKIN	10	15
SENSITIVE SKIN	12	5
EXPANDED CAPILLARIES AND INFLECTIONS	12	5

8/MAKE SURE STEAM IS GOING FROM CHEEK TO TOP OF THE FACE TO FULLY COVER THE FACE.

PRECAUTIONS :

1/TO RELEASE THE WATER JAR FOR CLEANING, UNSCREW THE KNOB AT THE BOTTOM AND HOLD WITH THE PLASTIC HANDLE TO AVOID BURNING.

2/TAKE OUT THE JAR AND WASH WITH DETERGENT IF IT IS DIRTY.

3/IF DEPOSIT IS FOUND AROUND THE HEATING ELEMENT, IMMERSE IN A JAR OF WATER WITH TWO SPOONFUL OF WHITE VINEGAR OVERNIGHT AND USE A METAL OR HARD BRUSH TO BRUSH AWAY THE DEPOSIT.

4/AFTER WASHING THE JAR, MAKE SURE THAT THE WHITE SILICONE GASKET IS IN PROPER POSITION WHEN YOU INSTALL THE WATER JAR TO THE ORIGINAL POSITON.

5/DO NOT PUT ANY AROMA SUBSTANCES OR OIL INTO WATER JAR BECAUSE THIS WILL CAUSE HOT WATER SPITTING WITH SERIOUS BURN TO THE CLIENTS.

6/THE MACHINE ITSELF IS NOT WATERPROOF. WHEN YOU ARE HANDLING THE WATER, DO NOT WET ANY OTHER PART ESPECIALLY THE CONTOL PANEL OF THE SYSTEM.

7/NEVER USE THE SYSTEM IN BATHROOM OR ANY WET PLACES.

8/YOU MAY ROLL THE SYSTEM SLOWLY FROM PLACE TO PLACE. WHEN THE STEAMER IS OPERATING, THE SYSTEM SHOULD NOT BE MOVED BY ANY MEANS.

9/THE OPERATING ANGLE FOR THE SYSTEM IS 270 DEG., PLEASE ROTATE SLOWLY.

10/IT IS RECOMMENDED TO DISGARD THE LEFT OVER DISTILLED WATER IN THE JAR AT THE END OF THE DAY'S OPERATION.

HI-FREQUENCY OPERATION MANUEL

ULTRAVIOLET IS TO STERILIZE AND CLOSE THE PORES.
INFRARED IS FOR ABSORPTION, INCREASE GLANDULAR ACTIVITIES.
INCREASE BLOOD CIRCULATION AND METABOLISM, ENHENCE
PENETRATION OF PRODUCTS.

1/SWITCH ON THE POWER SWITCH FOR HI-FREQUENCY.

2/PUT THE APPROPRIATE GLASS ELECTRODE TO THE CONNECTED
HANDLE. THE METAL PART OF THE ELECTRODE IS INSERTED INTO THE
HOLE.

3/INCREASE INTENSITY GRADUALLY BY THE KONB/DIAL TO
APPROPRIATE LEVEL TILL A SPARK IS SEEN OR CLIENTS FEELING OF
IT.

4/APPLY THE GLASS ELECTRODE OVER THE TREATMENT AREA.

5/TO CHANGE THE ELECTRODE BY STOPPING ANY ACTIVITY, ADJUST
TO ZERO INTENSITY FIRST. PULL OUT THE ELECTRODE GENTLY FROM
THE HANDLE. CAUTIOUS, ELECTRODE MAY BE BROKEN IF THE WAY OF
PULLING OUT IS WRONG. REPEAT PROCEDURE #2 AND #3.

6/THE FUNCTIONS OF DIFFERENT ELECTRODES:
MUSHROOM IN ULTRAVIOLET: STERILIZE AND CLOSE PORES.
ROD, SPOON AND J-ROD IN INFRARED: INCREASE GLANDULAR
ACTIVITIES, STIMULATES CIRCULATION OF BLOOD AND METABOLISM,
HELPS IN DEEPER PENETARTION OF PRODUCTS APPLIED.

HI-FREQUENCY PRECAUTIONS:

1/CLIENTS BEING TREATED MUST NOT WEAR ANY JEWELRIES OR IN CONTACT OF ANY METAL OBJECTS TO AVOID ELECTRICAL SHOCK.

2/CLIENTS MUST NOT BE PREGNANT OR EQUIPED WITH A PACEMAKER.

3/DO NOT USE A LOTION WITH ALCOHOL BASE TO-GETHER WITH HI-FREQUENCY TREATMENT. IF THIS IS NECESSARY, TREAT FIRSTLY WITH HI-FREQUENCY AND THEN APPLY THE ALCOHOL BASE LOTION OVER THE AREA.

4/THE GLASS ELECTRODE SHOULD BE CLEANED AND STERILIZED AFTER EACH USAGE.

5/DO NOT TOUCH OR REMOVE THE P.C. BOARD IN THE SYSTEM UNDER ANY CIRCUMSTANCES.

6/DO NOT USE THE HI-FREQUENCY WITHIN 2 METER/YARDS OF ACCESSABLE GAS.

VACUUM AND SPRAY:

VACUUM:

THIS SUCTION ACTS LIKE A MINI-VACUUM CLEANER FOR THE FACE. THE DIRTS AND GREASE IN THE PORES ARE BEING CLEANED AND THIS SUCTION ALSO DRAW BLOOD TO THE SURFACE OF FACE/SKIN TO INCREASE BLOOD CIRCULATION.

OPERATING PROCEDURES:

1/CONNECT THE SUCTION HOSE TO THE OUTPUT UNDER MACHINE. CONNECT THE OTHER END TO THE DESIRED GLASS VENTHOSE/TUBING.

2/SWITCH ON THE POWER ON THE CONTROL PANEL. THE RED LIGHT SHOULD BE ON BY NOW.

3/ADJUST THE STRENGTH OF SUCTION BY THE KONB/DIAL. USUALLY YOU CAN USE MAXIMUM.

4/USE YOUR FINGER TIP TO CONTROL THE STRENGTH AND MOVEMENT OF THE GLASS VENTHOSE.

5/CLEAN AND STERILIZE THE GLASS VENTHOSES/TUBINGS AFTER EACH USAGE.

6/SWITCH OFF THE VACUUM IMMEDIATELY AFTER USE TO AVOID ANY OBJECTS BEING SUCKED INTO THE MACHINE CAUSING DAMAGES.

PRECAUTIONS:

1/THE GLASS VENTHOSES HAVE A HOLE FOR THE FINGER TIP TO CONTROL THE VACUUM SUCTION. THERE WILL NOT BE ANY SUCTION IF THE HOLE IS LEFT OPEN.

2/IT IS RECOMMENDED TO APPLY A THIN FILM OF CLEANSING LOTION OVER THE SKIN TO ACT AS BUFFER AND ALLOWING THE GLASS VENTHOSE TO MOVE OVER THE SURFACE WITHOUT HURTING THE CLIENT.

3/FOR DRY,AGED OR THIN SKIN, USE LIGHT SUCTION. FOR OILY SKIN, USE MORE STRENGTH. THE STRENGTH OF SUCTION MAY BE TESTED BY THE INNER ARM OF THE ESTHETICIAN.

4/MACHINE IS NOT WATERPROOF, DO NOT USE IN BATHROOM OR WITHIN 2 METER/YARD OF ACCESSABLE GAS.

SPRAY:

SPRAY IS FOR CLEANSING OF SKIN'S SURFACE, COOLING AND SOOTHING. THE IMPACT OF SPRAY IS TO FLUSH OUT THE PORES ESPECIALLY AFTER SUCTION AND SQUEEZING OF BLACKHEADS AND PIMPLES.

OPERATING PROCEDURES:

- 1/CONNECT THE SPRAY HOSE TO THE CAP OF SPRAY BOTTLE FILLED WITH DESIRED LOTION.
- 2/SWITCH ON THE POWER OF VAC/SPRAY, THE RED LIGHT SHOULD BE ON.
- 3/ADJUST THE STRENGTH BY THE KONB/DIAL. USUALLY TO MAXIMUM.
- 4/PRESS FINGER TIP OVER THE TOP OF THE BOTTLE CAP AND SPRAY WILL START.

PRECAUTIONS:

- 1/TWO BOTTLES OF LOTION, ONE FOR AGED AND DRY SKIN, THE OTHER FOR NORMAL OR OILY SKIN.
- 2/DO NOT FILL TO THE TOP OF BOTTLE, TWO THIRDS ONLY TO OPERATE BEST.
- 3/DO NOT OPEN THE SYSTEM, WILL VOID WARRANTY.

GALVANIC:

THE GALVANIC CURRENT IS USED TO PICKED UP PIGMENTS OR METAL IONS LEFT BY COSMETIC AND INTRODUCING THE NUTRIENTS OF SKIN PENETRATING BY MEANS OF CURRENT INDUCTION.

IONTOPHORESIS : INTRODUCE WATRE SOLUBLE PRODUCTS INTO SKIN
PHORESIS : FORCE PHORESIS-CHEMICAL SOLUTIONS INTO BROKEN SKIN.

CATAPHORESIS : USING THE POSITIVE POLE TO INTRODUCE ACIDIC SOULTION INTO THE SKIN.

ANAPHORESIS : USING THE NEGATIVE POLE TO INTRODUCE ALKALINE SOLUTION INTO THE SKIN.

POSITIVE POLE IS USED FOR:

- 1/CLOSE THE FOLLICLES(PORES)AFTER FACIAL TRETAMENT.
- 2/DECREASE REDNESS AS IN MILD ACNE.
- 3/PREVENT INFLAMMATION AFTER COMEDONE AND BLEMISH TREATMENT.
- 4/TO FORCE ACIDIC SOLUTIONS SUCH AS ASTRINGENT INTO SKIN.

NEGATIVE POLE IS USED FOR:

- 1/TO STIMULATE THE CIRCULATION OF BLOOD TO DRY SKIN.
- 2/TO FORCE DISINCRUSTATION(ALKALINE) LOTION INTO SKIN.

OPERATING PROCEDURES:

1/USE A THIN LAYER OF COTTON TO APPLY THE LOTION OVER THE FACE OR AREA TO BE TREATED.

2/CONNECT THE CHROME ROD TO THE BLACK CORD AND ASK CLIENT TO HOLD IT TIGHT FOR GROUNDING. IT IS RECOMMENDED TO WET THE HOLDING HAND FOR BETTER CONDUCTIVITY.

3/SCREW THE DESIRED METAL ACCESSORY TO THE HEAD OF THE IVORY COLOR PLASTIC HANDLE.

4/SWITCH IONO KEY TO START AND THE RED LIGHT IS ON.

5/SELECT THE DESIRED POLARITY +VE OR -VE ON THE CONTROL PANEL.

6/ADJUST THE INTENSITY TO A COMFORTABLE LEVEL, USUALLY LESS THAN 3 mA.

PRECAUTIONS:

1/DO NOT USE GALVANIC CURRENT OVER AN AREA WITH LOT OF BROKEN CAPILLARIES. CLIENT SHOULD NOT WEAR ANY METAL OBJECTS OR JEWELRY.

2/THE ELECTRODE SHOULD BE KEPT IN TOUCH WITH SKIN ALL THE TIME. AVOID ON AND OFF TOUCHING WITH THE CURRENT IS ON, THIS WILL CAUSE UNCOMFORTABLE FEELING TO THE CLIENT.

3/MOVE THE ELECTRODE SLOWLY.

4/ALWAYS MAINTAIN GOOD AMOUNT OF FLUID OVER THE TREATING AREA TO ENSURE GOOD CONDUCTIVITY FOR THE GALVANIC CURRENT.

5/IF YOU NEED TO CHANGE THE TREATMENT AREA OR THE METAL ACCESSORY, STOP THE GALVANIC FIRST AND CHANGE, THEN SET ZERO AND START AGAIN.

6/IN CASE CURRENT IS OVER 3 mA, THE SYSTEM WILL STOP AUTOMATICALLY DUE TO SAFETY REASON. NOW YOU HAVE TO SET ZERO FIRST AND START AGAIN.

7/DO NOT REMOVE THE P.C. BOARD, SYSTEM IS NOT WATERPROOF. DO NOT USE IN BATHROOM OR WITHIN 2 METER/YARD OF ACCESSIBLE GAS.

BRUSHING:

THE BRUSHING FUNCTION IS TO PEEL AWAY THE DEAD SKIN CELLS AND REMOVE OF DIRT ON THE SURFACE OF SKIN. FACE IS VAPORIZED WITH STEAM AND MASSAGE WITH WARM TOWEL TO SOFTEN THE DEAD SKIN CELLS TO BEGIN. PRIOR TO BRUSHING, THE FACE SHOULD BE CLEANED WITH VISCOSE SPONGE OR COTTON PADS.

OPERATING PROCEDURES:

1/SWITCH ON THE POWER.

2/SELECT THE DESIRED BRUSH AND PLUG INTO THE BRUSH HANDLE SECURELY WITH A "CLICK" SOUND.

3/SWITCH ON THE BRUSH KEY TO START. ADJUST THE ROTATING SPEED BY THE KNOB/DIAL. SLOW IS FOR DRY AND SENSITIVE SKIN AND FAST IS FOR THICK SKIN.

4/THE "CHANGING" SWITCH IN THE HANDLE WILL ALTER THE ROTATING DIRECTION TO OPPOSITE IMMEDIATELY.

5/THE PEELING PROCEDURE STARTS FROM THE NECK TO THE JAW, CHIN, CHEEK, UPPER LIPS NOSE AND FOREHEAD.

6/IF YOU NEED TO CHANGE THE BRUSH, SWITCH OFF THE BRUSHING FUNCTION, CHANGE THE BRUSH AND START AGAIN.

7/BIG WOOL BRUSH--FOR CHEEK, CHIN AND FOREHEAD.
SMALL WOOL BRUSH--FOR EYELIDS AND NOSE.
SPONGE BRUSH--FOR SENSITIVE SKIN
PUMIC BRUSH--FOR DEAD SURFACE AND COARSE SKIN.

PRECAUTIONS:

1/FOR SLIGHTLY BROKEN, CAPILLARIES AREA, BRUSHING SHOULD BE VERY LIGHT. FOR SEVERE BROKEN SURFACE, NO BRUSHING SHOULD BE USED.

2/DURING APPLICATION, THE BRUSH SHOULD BE ALWAYS VERTICAL TO THE FACE.

3/AFTER EACH TREATMENT, THE BRUSH SHOULD BE CLEANED THOROUGHLY AND SOAP IN ALCOHOL FOR TWENTY MINUTES FOR STERILIZING. AFTER THAT, DRY AND PLACED IN STERILIZER..

MAGNIFYING LAMP:

OPERATING PROCEDURES:

1/PLUG THE POWER CORD TO THE BASE.

2/HOLD THE SWITCH FOR 5 SECONDS FOR THE LAMP TO GO ON.

3/ADJUST THE LAMP TO THE DESIRED ANGLE.

4/IF THE LAMP IS LOSE AND FALLING DOWN, TIGHTEN THE BUTTERFLY NUT ON THE SIDE TO SECURE IT AGAIN.

5/SWITCH OFF WHEN FINISH.

LIST OF ACCESSORIES:

1/HI-FREQUENCY HANDLE ONE SET WITH CORD.

2/FOUR PIECES OF HI-FREQUENCY ELECTRODES: MUSHROOM, ROD, SPOON AND J-DOT SHAPE.

3/ONE SET OF BRUSH HANDLE WITH CORD.

4/FIVE DIFFERENT BRUSHES, ONE LARGE WOOL BRUSH, ONE SMALL WOOL BRUSH, ONE SPONGE BRUSH, ONE LARGE PUMIC BRUSH, ONE SMALL PUMIC BRUSH.

5/TWO SPRAY BOTTLE.

6/ONE SET OF THREE PIECES OF GLASS VENTHOSES FOR VACUUM.

7/ONE GALVANIC CORD IN RED/BLACK COLOR.

8/ONE CHROME ROD FOR GROUNDING OF GALVANIC. ONE GALVANIC BULLET HEAD, ONE FLAT HEAD, ONE ROLLER AND ONE COTTON CLAMP.

9/ONE WOODS LAMP.